

Hydrogel Dressing Hemostatic Agent

عامل مرقئ هيدروجيل (موقف لنزيف الدم)



About the Product: This medical device is a non-woven cloth impregnated with a multidrug formulation. The napkin is folded in several layers and packed in a sealed individual package made of multilayer material. The packaging prevents the napkin from drying out and keeps the sterility of the product during the entire shelf life. The product is intended for use both in hospitals and at home.

Size: 18cm x 13cm

Sterile/Non-Sterile: Sterile

Purpose: Intended for use as a local hemostatic agent

Indications for use: First aid to stop external capillary bleeding, local treatment of abrasions, cuts, bruised, scalped, lacerated wounds.

Side Effects: Allergic skin reactions are possible, in which the use of the product should be discontinued.

Application Procedure: Before using EversLife-Gemo, the wound should be cleaned of foreign bodies and wound discharge, open the bag with a napkin along the notch, remove the napkin from the bag with tweezers or fingers, place the napkin on the wound surface so that it covers the entire surface of the wound, protruding beyond its edges by 1-2 cm and model according to the wound. After applying to the wound, the napkin is fixed with a bandage of a plaster type, a gauze bandage. Re-dressings are carried out if necessary, after 1-3 days.

Ingredients: Chitosan, gelatin, glycerin, aminocaproic acid, calcium gluconate, lidocaine, chlorhexidine, water, nonwoven material

Disposal: With household waste

Storage: In a dry place, protected from direct sunlight and sources that emit heat, at a temperature not exceeding +25 °C. Keep out of reach of children.

Series and date of manufacture: Indicated on the seal.

Shelf Life: 2 years

Manufacturer: LLC "EVTEKS", St. Timiryazava, 1, Office 18, 153025, Ivanovo City, Russia.

Exclusive Agent in MENA Region: Creative Pharma for Medical Supplies L.L.C.

Abu Dhabi, UAE. Tel.: 026278229, Email: info@creative-medicals.com

Evers
Group
Rus

**Manufacturer of innovative
and traditional medical devices**

Creative Pharma for Medical Supplies LLC
220 Hamdan Bin Mohammed Street,
Al Hson Tower, 3rd Floor, Office 302,
Abu Dhabi, U.A.E
02 627 8229

#Sterile #TM Evers Life

Hemostatic agent. “EversLife-Gemo”, hemostatic wipe

Modern medical hydrogel medical devices based on chitosan-gelatin polyelectrolyte complex for first aid.

Hemostatic action.

**The medical device accelerates blood clotting,
stops external capillary bleeding.**

Indications for use.

**Local treatment of abrasions, cuts, bruised,
scalped, lacerated wounds.**

Pain reliever and antimicrobial action.

**Ensured by the presence of local anesthetic - lidocaine hydrochloride and
antiseptic - chlorhexidine bigluconate.**

Hemostatic Wipe

- **Stops capillary bleeding**
- **Cleans the wound of germs**
- **Relieves pain**

Size	Qty per box	Qty per carton
18 x 13 cm	30 pcs	600 pcs
24 x 24 cm	40 pcs	240 pcs

